Quality made in Germany

Das Original

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Dental Instruments for rodent and rabbits

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### Quality means erbrich

For over 40 years now erbrich instruments GmbH have been producing and distributing surgical instruments for all medical disciplines as well as for the current extent of the veterinary field.

Our first class quality instruments have gained widespread recognition which helped in establishing an excellent reputation worldwide. This quality, combined with the innovation and reliability manifest in our complete programme, make erbrich-instruments GmbH an important partner in the field of medical technology.

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#### Rodent mouth gag (Mouth opener)

The rodent mouth gag provides a safe means of holding open the jaws of virtually all rodent and rabbit species. The wide range of adjustments ensures that the instrument can be adapted to the particular anatomical conditions, while also enabling the veterinary surgeon to adopt a comfortable posture. Tooth trimming is easy to perform on sedated animals even for a single operator because the animal is secured in the optimal position.

Simultaneous pressure on the upper and lower jaw minimises the risk of injury. Even in rodents with short, missing or malpositioned teeth, the jaws are easily opened, since the mouth opener is positioned behind the teeth. At the same time, this technique gives a better view of the oral cavity and also makes it easier to use the cheek dilator and other instruments for the dental treatment of rodents.

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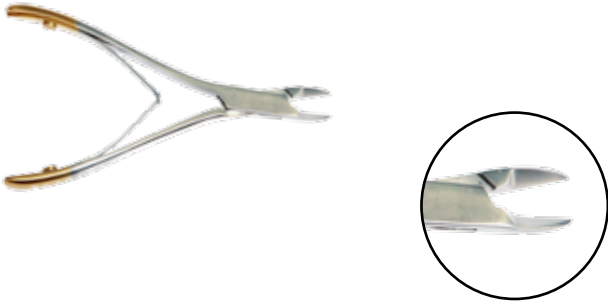
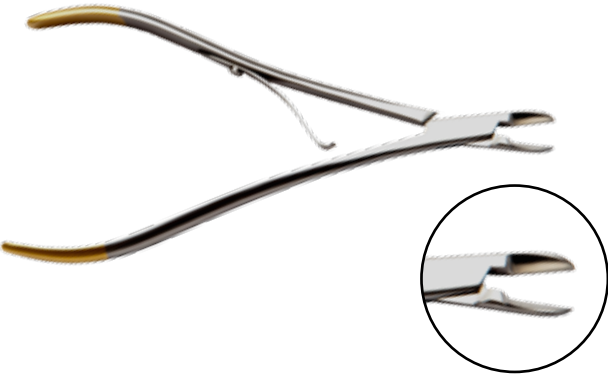
### Trimmer for incisors

This trimmer is designed for quick and easy trimming of malpositioned or excessively long rodent teeth. The cutting edges are shaped so as to ensure that uniform pressure is applied over the entire cross-section of the tooth at the moment of cutting, thus greatly reducing the risk of splitting.

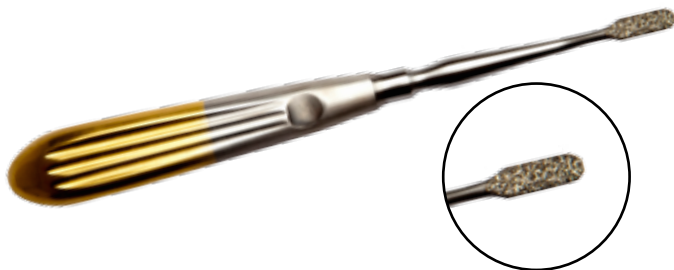
The long lever arm allows thick rabbit teeth to be trimmed with a minimum of force.

### Trimmer for molars

This trimmer enables the tips of molars to be removed quickly and easily. The risk of injury to the tongue and the cheek mucosa is minimised because the trimmer only has one cutting edge. The compact design allows very good visibility in the oral cavity.


The slender but extremely robust and sharp cutting edge also makes light work of trimming tooth protuberances on the rear molars.

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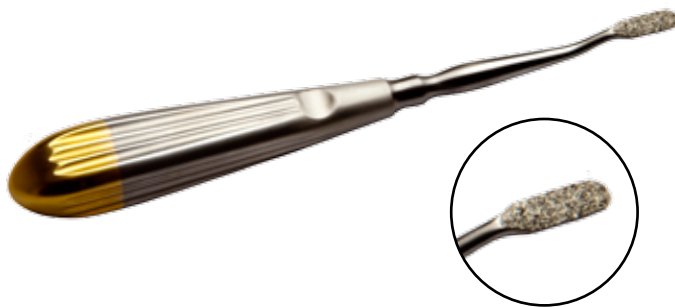### Tooth files (coarse and fine)

Both tooth files have a diamond-coated abrasive surface, which allows considerably more uniform trimming of molars than is possible with conventional tooth rasps. The short handle and small abrasive surface significantly reduce the risk of injuring the blood vessels located in the angle of the mandible.

The coarse file is used to quickly trim any angular tooth protuberances that are difficult to reach with the trimmer. The fine file is used for smoothing off after trimming the tooth tips, and for removing small protuberances.

Both files are also well suited to trimming deformed bird and tortoise beaks.

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erbrich-instrumente GmbH