erbrich hippo-dent®

Pferdezahn-Behandlung
komplett

Dental Treatment of Horses
complete

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The **erbrich - Instruments GmbH** produces and distributes since about 35 years surgical instruments for all branches as well as instruments for the veterinary medicine.

Due to their first class quality **erbrich - Instruments** have an excellent reputation worldwide.

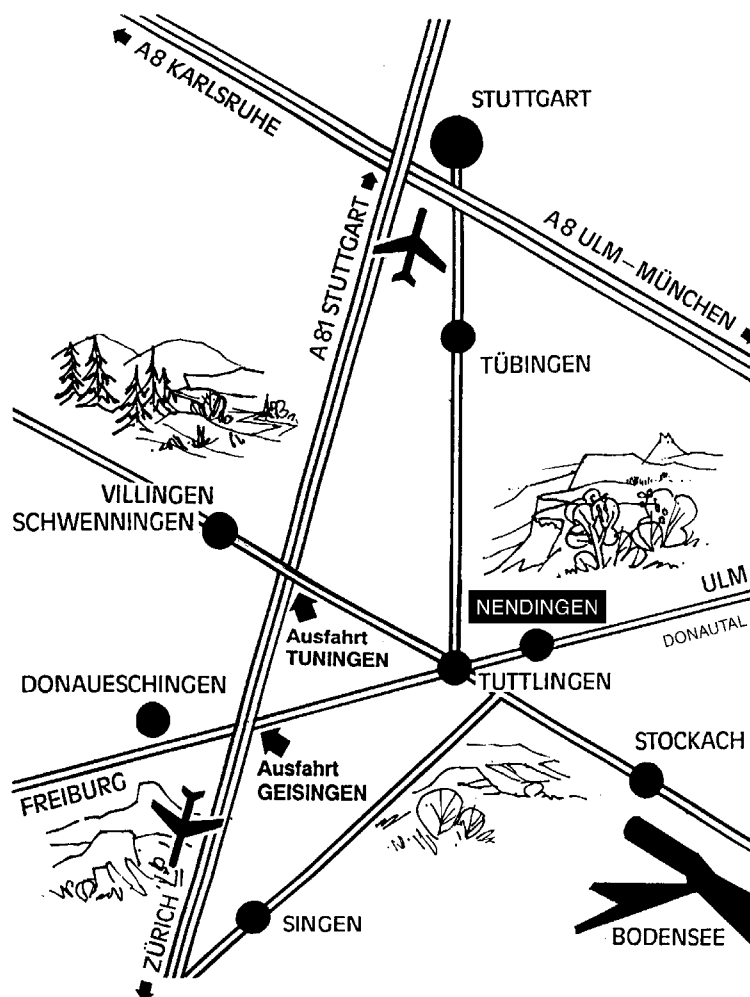
In addition to the quality, the innovation, the reliability as well as the complete program are reasons to make **erbrich - Instruments** as an important partner in the medical technology

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The development of instruments for the veterinary treatment of domesticated animals has ancient roots. It goes without saying, that progress in technology over the years led to an improvement of quality, material and enhanced specification of the instruments for the appropriate purposes.

However, it was only at the beginning of the thirties that Prof. Dr. Erwin Becker gained a reputation as an equine dentist. Due to the Situation caused by the lack of food for army horses during the second world war he began to develop modern equine dentistry together with the appropriate instruments. He was a professor at the Institut for Radiology, Animal Dentistry and Hoof Science in the veterinary faculty of Berlin University. Even measured by modern Standards, many of his publications underline and demonstrate his enormous knowledge and his excellent ideas and methods of therapy for modern equine dentistry.

Previous as well as current studies prove that up to 97 % of all horses reveal pathological changes of their dentition. Primarily, these are sharp edges and hooks of the molars. Furthermore, we also notice dentition problems (retained milk teeth), malocclusions (wavy dentition, parrot mouth etc.), abnormalities of the incisivi as well as surgical problems (fractures, abscesses of the root tips, different types of injuries). In addition, we observe a variety of other and associated illnesses (colic, sinus empysema etc.). The pathology and disorders of teeth in horses are manifold, in many cases, however, they are due to the fact that horses are fed with inappropriate, mechanically (pressed) and thermally-treated (roasted) commercial diets. Also the long feeding intervals (twice a day) and the lack of roughage as well as insufficient grazing contribute to the development of dentition problems.

The necessity and increasing interest of equine practitioners in modern equine dentistry result in a requirement of new professional instruments.

The new interest was the reason to develop in close collaboration with the **erbrich-Instrumente GmbH.**, Tuttlingen / Germany, modern instruments for dental treatment of horses for the practitioners.

In the days of E. Becker horses used to be tied up in stands but nowadays the treatment of horses is performed standing up after applying appropriate sedatives or under general anesthesia.

Even in those days equine dentists already had a **source of light** available for the examination of the oral cavity. Today various models with differing quality of light are on the market.

There are several devices for the opening and the examination of the oral cavity such as **mouth openers** (Stelzer, acc. to Bayer etc.), **full mouth specula** (acc. Prof. Dr. Becker, modified by Dr. Stelzer) and a **thumbspeculum** (Picture 1).

Common to all these instruments is a more or less clear

Bild 1 - erbrich-Artikel-Nr. **65E-12.00**
Picture 1 - erbrich-article-No. **65E-12.00**

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view of the oral cavity. For examining the oral cavity the thumbspeculum is adequate whereas a full mouth speculum should be used for the treatment of the cheek teeth and incisivi.

All instruments for equine dentistry are manufactured and sold by **erbrich-Instrumente GmbH** are made of highclass surgical steel, they are of excellent quality and have been tested thoroughly in practice by the author.

Dr.Becker developed an electric tooth-grinding-machine with a flexible driving shaft, different corundum grinding heads and various hand parts. This machine is heavy and only for restricted use in stables. The new hand-grindingmachine (drill drive with rotating grinding corpuses) developed by **erbrich-Instrumente GmbH** is distinguishes itself by an aluminium shaft (saving of weight), Cardan drive (no loss of power through belts) slipper clutch (for limiting the affect of power put on tooth), a small grinding wheel case as well as alternatively a diamond-coated or hard metal-grinding disk. (Picture 2 and 3)

Bild 3 - erbrich-Artikel-Nr. **65E-038.23** und **.11**
Picture 3 - erbrich-article-No. **65E-038.23** and **.11**

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There have always been several different types of forceps for the extraction of incisivi, cheek teeth as well as retained milk teeth. All of these instruments however, have in common that they are heavy, difficult to handle and often of poor quality (development of rust, torsion inflexibility ect.). By developing and improving established and old extraction instruments, a comprehensive **set of forceps** is now available (extraction forceps, milk incisivi forceps, milk cheek teeth forceps, big universal forceps with locking device and as well a hypomochleon as abutment). (Picture 4-7)

Bild 4 - erbrich-Artikel-Nr. **65E-044.44**
Picture 4 - erbrich-article-No. **65E-044.44**

Bild 6 - erbrich-Artikel-Nr. **65E-042.40**
Picture 6 - erbrich-article-No. **65E-042.40**

Bild 5 - erbrich-Artikel-Nr. **65E-040.23**
Picture 5 - erbrich-article-No. **65E-040.23**

Bild 7 - erbrich-Artikel-Nr. **65E-046.49**
Picture 7 - erbrich-article-No. **65E-046.49**

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Old **teeth cutters** for shortening excessively long teeth, e.g. the P2 in the upper jaw, are no longer used nowadays as they no longer conform to modern requirements. The blades of the two **cutters** offered in this program are made of specially heat-treated refined steel and formed in such a way that the tooth is not grasped at one point but on an imaginary cutting line, thus avoiding a crushing or breaking of the tooth. The transmission reduces the strength required by the surgeon to a minimum. Two models are available: one with a straight jaw and the other with an angular jaw for a better grasp on the P2 in the upper jaw. (Picture 8)

The **tooth chisel** is used for chipping off edges of the cheek teeth and loosening milk cheek teeth. Its hardened cutting edge and its easy-to-hand handle make it a very matured instrument. (Picture 9)

Bild 9 - erbrich-Artikel-Nr. **65E-030.00**
Picture 9 - erbrich-article-No. **65E-030.00**

Professional removal of so-called "wolf teeth" is an easy to perform task using the "**Wolf teeth extractor set**" Especially emphasized have to be the cutting sleeves (which are pushed over the wolf's tooth) available in four different sizes, so that injuries of the gingiva are reduced to a minimum. At the end of the handle there is a semicircular blade for removing remains of roots and for curettage of the alveole, if necessary. (Picture 10)

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The set of hardened **hollow chisels** (5 models from 6 to 14 mm) facilitates the loosening and levering of incisivi and cheek teeth (including fracture remains) and supplements the set of surgical instruments. (Picture 11 and 12)

Bild 12 - erbrich-Artikel-Nr. **65E-067.06/14**
Picture 12 - erbrich-article-No. **65E-067.06/14**

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Various **hand rasps** coated with hard metal granulate (bent hand rasp, as well a rasping system with two handles and three attachments) are available for manually shortening and finishing the pretreatment by machine. Furthermore, an incisivi rasp of standard steel is available. (Pictures 13 - 16)

Bild 15 - erbrich-Artikel-Nr. **65E-035.00**
Picture 15 - erbrich-article-No. **65E-035.00**

Bild 14 - erbrich-Artikel-Nr. **65E-035.00**
Picture 14 - erbrich-article-No. **65E-035.00**

Bild 16 - erbrich-Artikel-Nr. **65E-035.00**
Picture 16 - erbrich-article-No. **65E-035.00**

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Dr. med. vet. Peter Stelzer

The surgical removal of cheek teeth is conventionally performed by trepanization. A more modern method, the buccotomy, is also becoming increasingly popular. However, the decision as to which operating technique should be applied depends on various factors and ought to be decided on individually. For both techniques, a set of surgical instruments is available (wound expander, periost scraper, trephine with three different crown attachments, two osteotomes of different width for accesses, hammer and a set of tooth stamps).

The interesting innovation is found in the **tooth-stampingset**. In the new models, the conventional square crosssection has been replaced by a round one. This prevents the stamp getting canted or fixed in the bone (reduced risk of injuries). The knurled handle results in stamps resting firmly in the hand.

The development of instruments for equine dentistry is carried on continuously since these instruments are tested every day by the author and thus can be changed or supplemented if necessary. We would appreciate to receive reports by colleagues on any personal experiences concerning treatment methods and use of instruments, and would be glad to help and offer our advice in case of any problems.

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65E-169.23

Working attachment

Reinforced, anodized aluminium shaft tube capable of withstanding extreme loads.
Knurled section for a sure grip
Rasp head made of high-alloy surgical stainless steel means no flaking chrome or nickel plating
Small head for easier access to posterior molars
Rasp head is protected by five hydrodynamic seals to prevent dirt from entering the gears
New gearwheel geometry ensures efficient power transmission, quiet operation, low vibration, less friction and a long service life
The gears are mounted in high-alloy bearings of the type used in the aircraft industry – a further guarantee of a long service life

Rasp disk

Main body made of hardened stainless steel for dimensional stability and to protect the gears
Optimised geometric rasp disk holder for power transmission with maximum efficiency
Diamond-coated as standard for long life - up to 300 treatment cycles
Grinding with the diamond coating produces a texture on the tooth surface which is comparable to natural wear
State-of-the-art coating technology prevents any flaking of the diamond coating

Revolutionary power transmission system

Newly designed connection between drive unit and attachment eliminates the need for a slip clutch – even simpler
Direct drive via unbreakable shaft with multiple bearings, so there is no belt to snap or slip:
no power loss, quiet operation, long service life

Speed control

12 speeds from 1100 to 2550 rpm (relative to the rasp disk) can be preselected and set for relaxed working
The unit can be set to suit the individual tooth hardness, which means shorter treatment times

Drive unit

Low-maintenance 650 W professional power drill
Drill stops automatically when carbon brushes are worn down
Replaceable 6 m rubber-sheathed power cable for greater freedom of movement and no twisting

Technical data

230 V AC, 50 Hz
Output 650 W
Speed 0–2550 rpm (relative to the rasp disk)
Doubly insulated motor – Class II
Free from interference
Conforms to CE marking requirements
Weight 2.300 g

Scope of supply:

65E-169.23, 65E-169.01, 65E-169.03,
65E-038.07, 65E-038.08, 65E-038.11

Accessories

65E-169.01
Spare diamond-coated disk
Long life (standard)

65E-169.02
Spare diamond-coated disk
High Speed Cutting

65E038.02
Spare diamond-coated disk
Long life (standard)
for EM2000

65E-038.09
Spare diamond-coated disk
High Speed Cutting
for EM2000

65E-169.03
Spare screw

65E-169.04
Special grease

65E-038.08
Screwdriver

65E-038.10
Spare 6 m cable

65E-038.11
Plastic carrying case for tooth grinding unit

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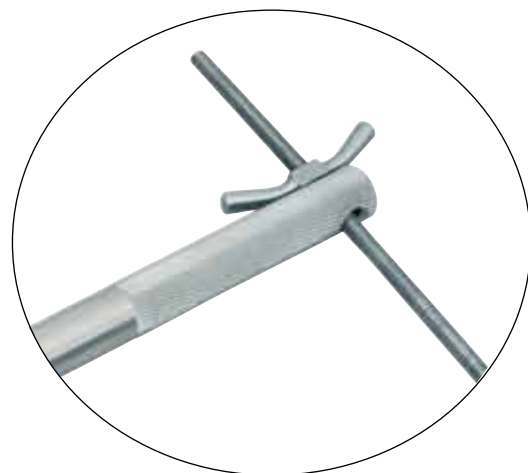
Its completely **new jaw design** makes the erbrich Universal Extraction Forceps still more reliable, even under most stringent conditions. A movable jaw ensures both easy and safe fixation of the tooth to be extracted.

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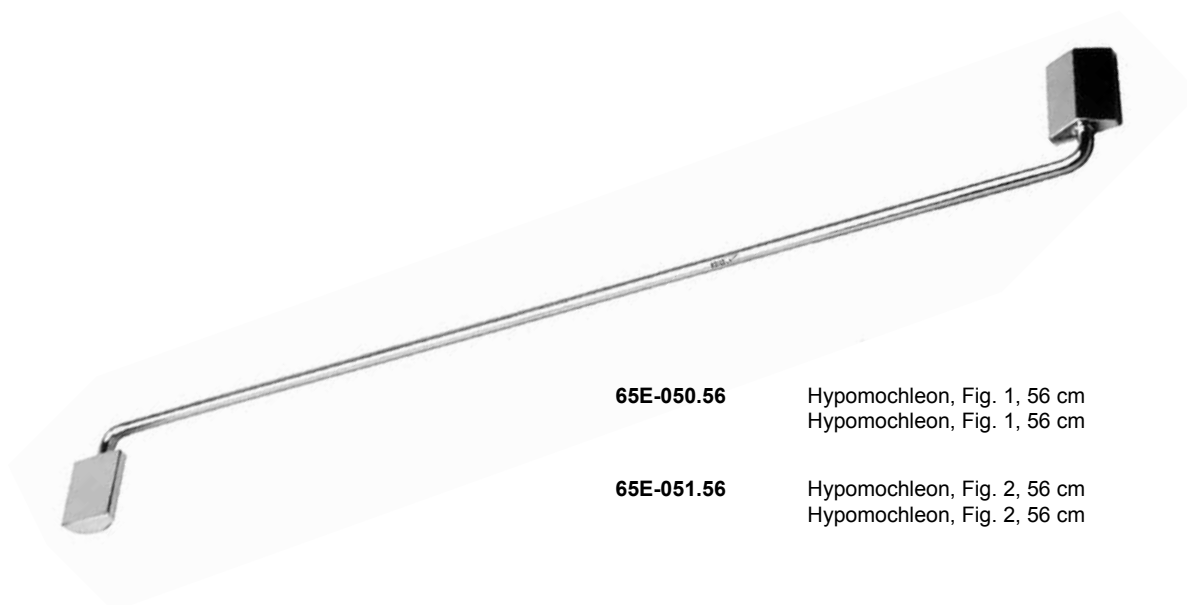
In order to further improve the fixing properties the extra large working surfaces of the jaws have been given a **particular structure**: The fixed jaw is provided with **longitudinal serration** while the movable jaw fitted with **transverse ribs**.

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Fixing is further facilitated by a **locking screw** with an **extra fast-running threaded rod**.

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Wolf Tooth Extractor- SET Dr. Stelzer, complete with 4 tops, in canvas roll

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erbrich-instrumente GmbH