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rostfreier Edelstahl

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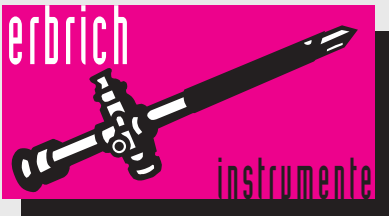
Tick remover

stainless steel

The bloodsucking tick transmits TBE and borreliosis

TBE (tick-borne encephalitis) and borreliosis can cause inflammations of the meninges, the brain, the nerves and of the medulla.

Generally the borreliosis virus will be transmitted one day after the tick bite. Because of that it is very important to remove the parasite immediately. In the process it is essential to consider not to crush the abdomen.

With the tick remover you're able to remove the tick safe and simple. .

This product is protected by letters patent. We point out explicit that the use of the tick remover happens at owner's risk. We exclude the liability for harms of any sort.

1.
Open the forceps by pushing the opening lever with the forefinger..

2.
Lead the short end below the abdomen of the tick, as close as possible to the skin of the host.

3.
Close the forceps by removing the forefinger from the opening lever.

4.
Now remove the parasite by extracting it. We recommend to make a rotary motion during the extraction.

5.
Keep the tick safe and bring it to your general practitioner. He is able to test the tick for borreliosis and TBE and he is also able to take the required actions if necessary.

erbrich - instrumente GmbH

Eugenstr. 33 • D-78532 Tuttlingen

Tel.: +49 74 61 / 96 57 30

Fax: +49 74 61 / 96 57 320

E-mail: erbrich-instruments@t-online.de

Website: www.erbrich-instruments.com